**Capillary-Blood Tubes:  
Test Procedures**

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**REAGENTS:**

Per tube (maximum): Acridine Orange (5.6  $\mu\text{g}$ ), Potassium Oxalate (0.3 mg), Sodium Heparin (3.8 USP units),  $\text{K}_2$  EDTA (0.44 mg), Agglutinating Agents and Stabilizers.

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## QBC<sup>®</sup> E-Z Prep<sup>™</sup> Capillary-Blood Tubes

### INTENDED USE:

For hematology determinations with capillary blood using QBC-series Analyzers. For *In Vitro* Diagnostic Use.

### REAGENTS:

Acridine orange (AO), potassium oxalate (KOX), sodium heparin, dipotassium EDTA, agglutinating agent, and stabilizers. See concentrations on vial label.

### STORAGE:

Keep tubes sealed in glass vial when not in use; remove one by one as needed. Do not remove desiccant strip from tube vial. Store vial sealed, in dark dry place at 60° to 90°F (16° to 32°C).

### STABILITY:

Printed date on vial label is expiration date of tubes in **unopened vial**. Tubes can be used for **30 days** after first opening of vial, provided expiry date is not exceeded. Record opening date of vial in space provided on label.

### CAPILLARY BLOOD COLLECTION:

#### WARNING:

BLOOD SPECIMENS MAY CONTAIN HEPATITIS B VIRUS OR HUMAN IMMUNODEFICIENCY VIRUS (HIV). HANDLE BLOOD AS POTENTIAL BIOHAZARD CAPABLE OF TRANSMITTING INFECTION. ALWAYS WEAR PROTECTIVE GLOVES.

Collect capillary blood directly from finger puncture into QBC tube according to techniques under PREPARATION AND TESTING. Be sure skin area is thoroughly wiped with antiseptic agent before puncturing with **sterile lancet**. Blood must be free-flowing and collected with minimum delay. QBC capillary-blood tubes are pre-coated with anticoagulant.

## PREPARATION AND TESTING:

Materials Provided: QBC E-Z Prep capillary-blood tubes with pre-assembled float and partially seated stopper; stylet wires (2) for repositioning float.

Materials Required: QBC-series hematology system (Reader and Centrifuge), lint-free wipers.

Operating Temperature Requirements: 68° to 90°F (20° to 32°C).


### CAUTION:

TO PROTECT FROM INFECTIOUS DISEASE, BE SURE TO WEAR LABORATORY GLOVES. USE CARE WHEN HANDLING GLASS QBC TUBES TO AVOID BREAKAGE AND POSSIBLE INJURY.


### Procedures:

1. INSPECT FOR CORRECT STARTING POSITION OF FLOAT AND STOPPER BEFORE FILLING BLOOD TUBE.


- Float** must be near stopper, away from orange coating area. See A. If necessary, *gently tap side of tube* to move float to proper position. DO NOT TAP STOPPER, since this may disturb correct seating of stopper. If float remains in orange coating area after tapping, use stylet provided with tubes to reposition float. Insert stylet through open end of QBC tube and push float to stopper end of tube.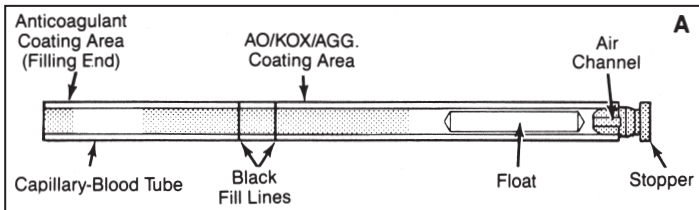


- Stopper** must be partially seated in tube so that both air channels in stopper are open when aspirating blood. See B.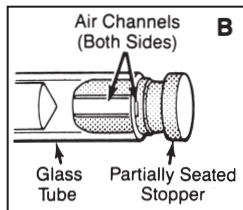


2. COLLECT FINGER BLOOD (C) into end of tube nearest two black lines, filling to level between black lines. Wipe outside surface of tube with lint-free tissue.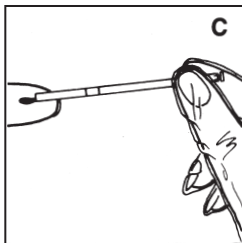

3. MIX BLOOD by holding tube nearly horizontal and rolling it between fingers several times to mix blood with anticoagulants (D).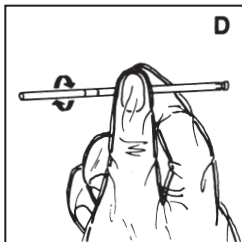

Turn tube around, tilt and roll tube between fingers while allowing blood to flow to orange-coated area (E). Roll tube between fingers at least 10 times or for at least 5 seconds to mix blood with AO/KOX.

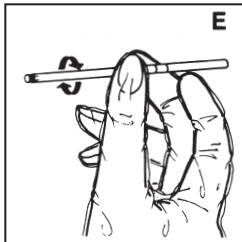4. SEAT STOPPER by holding tube horizontally and pushing stopper firmly against clean, vertical surface until stopper is fully seated.
5. CENTRIFUGE FOR 5 MINUTES by placing tube(s) on rotor of QBC centrifuge and spinning tubes down according to procedures in applicable manual. After 5-minute centrifugation, PROMPTLY REMOVE TUBES. Place tubes stopper-down in work station, or read. [Note: Centrifuged blood is stable for 4 hours before reading, provided tubes are stored as noted above, away from heat and intense light.]

6. READ AND OBTAIN TEST VALUES by inserting tube in QBC Reader and testing in CAPILLARY MODE according to procedures in applicable operator's manual.

Depending on model of QBC analyzer system, E-Z Prep capillary blood tubes will provide the following quantitative hematology values: hematocrit, hemoglobin, mean corpuscular hemoglobin concentration, platelet count, white blood cell count, granulocyte count (% and number), and lymphocyte/monocyte count (% and number).

**PERFORMANCE:**

See applicable QBC operator's manual for information on precision and accuracy of capillary-blood hematology tests.

**CALIBRATION CHECK:**

Perform daily using Calibration Check Tube(s) supplied with QBC System.

**ORDERING INFORMATION:**

- QBC E-Z Prep Capillary-Blood Tubes: 100 tests—Cat. No. 424641
- QBC E-Z Prep Venous-Blood Tubes: 100 tests—Cat. No. 424640

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**QBC, E-Z Prep** is a trademark of QBC Diagnostics, Inc.
Copyright © 1992 by QBC Diagnostics, Inc.
Copyright © 1992 von QBC Diagnostics, Inc.
Copyright © 1992 QBC Diagnostics, Inc.
Copyright © 1992 della QBC Diagnostics, Inc
Copyright © 1992 por QBC Diagnostics, Inc.