

---

---

## Certificate of Decontamination

---

---

This form must be completed, signed and returned to Drucker Diagnostics by the customer (a) **before** transporting, by any means, a product for repair or service, and (b) prior to any on-site service to be performed by Drucker Diagnostics.

A completed copy of this Certificate is required to be attached to the instrument or part itself as well as the outside of the packing container when transporting to the facility. Please see page 2 for decontamination instructions.

Equipment being returned:

Model No. \_\_\_\_\_

Serial No. \_\_\_\_\_

Reason for its return:

\_\_\_\_\_  
\_\_\_\_\_

RMA or RGA Number: \_\_\_\_\_

This equipment (check all that apply):

**New Product**

Never used

**Biohazards**

Not used

Used, but decontaminated with:

\_\_\_\_\_

**Hazardous Chemicals**

Not used

Used, but decontaminated with:

\_\_\_\_\_

**Radioactive Materials**

Not used

Used, but decontaminated with:

\_\_\_\_\_

**I certify that the equipment described above has been thoroughly cleaned and decontaminated of all chemical, biological and radioactive contaminants and I also certify that the returned unit is safe for unprotected human contact.**

By: \_\_\_\_\_  
*Signature*

\_\_\_\_\_ *Print Name*

Title: \_\_\_\_\_

Date: \_\_\_\_\_

Company: \_\_\_\_\_

Address: \_\_\_\_\_  
\_\_\_\_\_

Phone: \_\_\_\_\_

E-Mail Address: \_\_\_\_\_

## Decontamination/Disinfecting Instructions for Centrifuges

- Unplug the centrifuge before disinfecting.
- Use appropriate personal protective equipment (PPE).
- Apply disinfecting solution with a towel or cloth. Do not submerge the machine in water or other cleaning solutions as this will cause damage to the unit.
- **ONLY isopropyl alcohol or a 10% (5500 PPM) bleach solution should be used to disinfect the centrifuge and its accessories. Wipe off thoroughly after use to prevent damage to the units.**
- All surfaces must be dried immediately after disinfecting.
- **TBQ GERMICIDAL PRODUCTS ARE NOT RECOMMENDED AS THEY MAY CAUSE DAMAGE TO THE CENTRIFUGE AND VOID THE WARRANTY.**
- Fully/partially halogenated hydrocarbons, ketones, esters, ethers, benzyls, ethyl benzenes, and all other chemicals not prescribed by the manufacturer should not be used as they may cause damage to the rotor chamber, rotor, tube holder, accessories, and centrifuge exterior.
- **Failure to decontaminate the centrifuge and/or provide the Certificate of Decontamination will result in a \$50.00 fee.**

## Shipping Instructions

- Place the machine inside a box with bubble wrap/packing peanuts.
- Display the RGA # provided to you on the outside of the box.
- Drucker Diagnostics, 200 Shady Lane, Suite 170, Philipsburg, PA 16866.

Please direct any questions to [customerservice@druckerdiagnostics.com](mailto:customerservice@druckerdiagnostics.com) or call us at 814-692-7661.